


# hCG COMBO

In-built positive and negative controls

Inverness Medical

## TESTPACK™+Plus

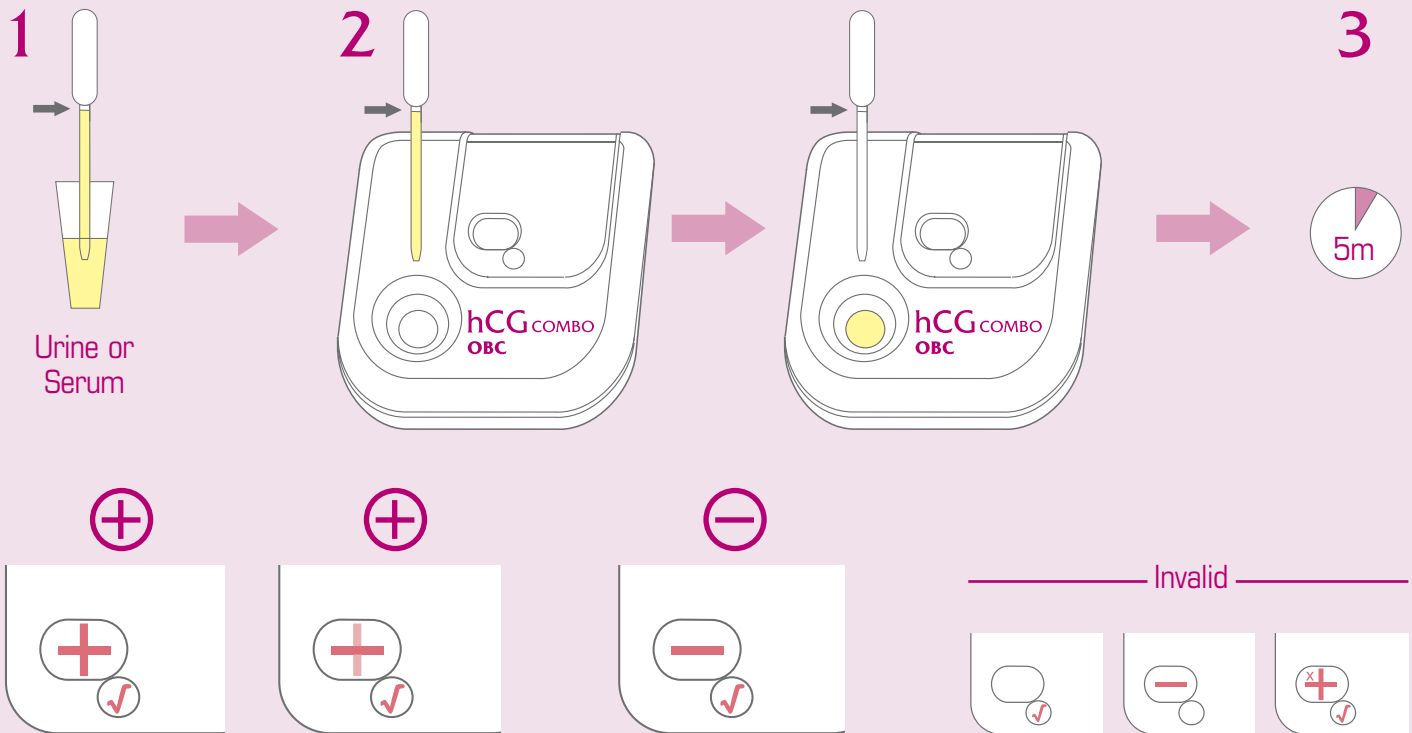


Immediately before use, remove test from foil pouch.

- 1 Draw urine or serum sample to line marked on pipette.
- 2 Dispense ENTIRE CONTENTS of pipette drop-wise into sample well.
- 3 Read result at 5 minutes.  
DO NOT interpret results after this time.

### Assay Check

MUST BE present for valid test results

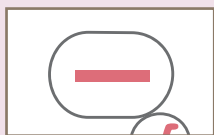

**The Minus Sign**

A control showing that the sample has run through the test correctly.



**The Positive Control**

Indicates the antibodies that detect hCG are working.

MUST NOT be present



**The Negative Control**

Indicates a non-specific entity in the sample that could cause a false positive result.

For a Positive Result



**The Patient Bar**

Indicates the presence of hCG in the sample.